

## Technical Data Sheet

### **Product: Alident CS05 Mouth Rinse**

Revision: 0 of 04.04.2016

**INGREDIENTI:** Aqua, Sorbitol, Polysorbate 20, Poloxamer 407, Aroma, Sodium Methylparaben, Limonene, Mentha Piperita Oil, Citric Acid, Cetylpyridinium Chloride, Sodium Ethylparaben, Sodium Fluoride, Sodium Saccharin, CI 42051.

**ASPETTO:** Clear solution, blue coloured.

**PROFUMO:** Peppermint

**TOSSICITA':** N.D.

**ATTIVITA':**

Alident CS05 mouthwash with Cetylpyridinium Chloride 0,05% is suggested for daily hygiene of mouth and gums, thanks to its active principle. Alident mouthwash CS 05 is a medical device suggested to decolonization of oral mucosa through physical cleaning. It inhibits MRSA propagation thanks to high efficient antimicrobial barrier effect.

**INDICAZIONI D'USO:**

For daily oral hygiene and disinfection.

**TOLLERABILITA':**

High tolerability.

**WARNINGS:** To be used only on undamaged mucous membranes. Do not use for infusion or injection. Do not swallow. In case of ingestion, rinse out the mouth, drink plenty of water and seek medical advice. If accidental contact with eyes occurs, wash immediately with plenty of water. If eye irritation persists, consult a specialist. Keep out of reach of children. The bottle must be closed immediately after each use. Use only unobjectionable and undamaged bottles. Do not use the product in case of hypersensitivity to any of the ingredients. Suspend the use of the product in case of mucosal irritation. Store in a cool and dry place, away from light and heat sources, at a temperature between 4°C and 30°C. Do not use after expiry date indicated on the bottom of the bottle.

**CONTROINDICAZIONI:** Do not use the product in case of hypersensitivity to any of the ingredients. Do not swallow.

**VALIDITA':** 36 months

**CONFEZIONI:** Bottle of 200 ml (in boxes 12 of bottles)

**MODALITA' DI CONSERVAZIONE:** Store in a cool and dry place, away from light and heat sources, at a temperature between 4°C and 30°C.