



## **PRODUCT INFORMATION**

### **ECG SPRAY (CONTI SPRAY)**

ContiSpray is a very good electrotransmission gel for ecg diagnostics. Thanks to its particular structure, it helps the probe moving, thus making the analyst's work easier. ContiSpray is watersoluble, drying-resistant, easily removable. ContiSpray doesn't stain, does not grease. It does not contain any active principles that might damage the electrodes.

#### **INSTRUCTIONS FOR USE**

- \* Spray is ready to use
- Clean and degrease the patient skin.
- Apply the desired amount of ECG. SPRAY
- After the procedure remove all residues of the spray with paper, TNT (Non-woven) or gauze moistened with water.
- \* Stop using the gel for hypersensitive patients, if skin reactions occur.

#### **WARNINGS**

- External use only.
- The product is not sterile.
- Do not apply spray on skin affected by abrasions, erythema or wounds.
- Close the bottle after use.
- \* May cause irritations or reddenings with allergic persons.
- \* Contact with eyes may cause irritation.
- \* Analyst with sensitive skin should wear protective gloves during the use.
- \* Do not use the spray after the "use by" date on the bottle.

**INGREDIENTS:** aqua, propylene glycol, sodium chloride, E131, E 102

#### **PACKAGING**

The product is supplied in the following packagings:  
250 ml (35 / box)

#### **DISPOSAL CONSIDERATION**

Product, waste: Chemicals must be disposed of in compliance with the respective national regulations.  
Packaging must be disposed in compliance with the country-specific.

#### **STORAGE**

Store at a dark place in room temperature. Keep container firmly closed, away from direct sunlight and sources of UV radiation.

#### **EXPIRY**

36 months from production date

---

**Ultragel Hungary 2000 Kft.**

✉ 1023 Budapest Bécsi út 4.  
📧 [info@ultragel.hu](mailto:info@ultragel.hu)

☎ +36 1 278 3050  
🌐 [www.ultragel.hu](http://www.ultragel.hu)