



SOL-M™ Blunt Fill Needle

SOL-M™ Blunt Fill Needle with Filter

Medication
Preparation




- To help reduce risk of needlestick injuries to healthcare workers when **drawing up** and **preparing medication** in compliance with Directive 2010/32/EU.
- **Prevention from sharp injuries** in hospitals and other health facilities.
- **Eliminating unnecessary use of sharps.**

● **Red** – Blunt Fill Needle



● **Purple** – Blunt Fill Needle with Filter

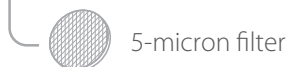

5-micron filter

● **Easy Identification** - Color-coded hub.

Blunt Fill Needle (BFN) for use with vials.

Blunt Fill Needle with 5-micron filter (BFNF) for use with glass ampoules.

● **Ease of Use** - Both needles have a conventional needle hub compatible with all standard Luer Lock- and Slip Tip- syringes.

● **Reduced Medicine Contamination** - The 5-micron filter in the hub of the BFNF means any glass particles larger than 5 micron are filtered out, preventing them from being drawn into the syringe prior to injection.

SOL-M™ Blunt Fill Needle

SOL-M™ Blunt Fill Needle with Filter



Usage Guidelines

Remove the Blunt Fill Needle from its package and connect to the syringe.

Remove the needle cap.

BFN - Insert the needle into the center of the vial stopper at a 90° angle. This will help reduce the risk of rubber fragments from the side of the vial stopper contaminating the medication.

BFNF - Draw up medication from a glass ampoule. Note: not for drawing up medications with molecules greater than 5 microns.

Once medication has been prepared and is in the syringe, ready for administration, remove the BFN/BFNF and dispose it into a sharps container.



The Blunt Fill Needle is for **single use only**. Reuse will dull the needle, remove the silicone coating and increase the chance of coring and rubber fragmentation.



Product Specification

Reference number	Product description	Gauge size	Outer diameter (mm)	Length	
				(mm)	(inch)
110023	SOL-M™ Blunt Fill Needle 18G*2"	18G	1.2 mm	50 mm	2"
110023F	SOL-M™ Blunt Fill Needle w/ Filter 18G*2"	18G	1.2 mm	50 mm	2"
110022	SOL-M™ Blunt Fill Needle 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"
110022F	SOL-M™ Blunt Fill Needle w/ Filter 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"
110021	SOL-M™ Blunt Fill Needle 18G*1"	18G	1.2 mm	25mm	1"
110021F	SOL-M™ Blunt Fill Needle w/ Filter 18G*1"	18G	1.2 mm	25 mm	1"

Note:

- Needle tube length tolerances based on ISO 7864 (dimensions in millimeters)

Nominal length of needle tube	25 to 39	40	> 40
Tolerance	+1.5 / -2.5	0 / -4	+1.5 / -2.5

CE-marked

All products with Medical Paper

Sales Unit: 100 pcs/Box; 1000 pcs/Case